**AT52203**  
**INSTRUKCJA UŻYTKOWNIA, ELEKTRYCZNY PODNOŚNIK PACJENTA**  
**USER MANUAL, ELECTRIC PATIENT LIFT**  
**GEBRAUCHSANWEISUNG, ELEKTRISCHER PATIENTENLIFTER**  
**NÁVOD K POUŽITÍ, ELEKTRICKÝ ZDVIHÁK PACIENTŮ**  
**NÁVOD NA POUŽITIE, ELEKTRICKÝ ZDVIHÁK PACIENTA**  
**MODE D'EMPLOI, LEVAGE ÉLECTRIQUE DU PATIENT**  
**GEBRUIKSAANWIJZING, ELEKTRISCHE PATIËNTLIFT**  
**MANUAL DE USO, ELEVADOR ELÉCTRICO DE PACIENTES**  
**MANUALE D'USO, SOLLEVATORE ELETTRICO PER PAZIENTI**  
**BRUKSANVISNING, ELEKTRISK PATIENTLYFT**

**MD**

## **PL**

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EN

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or the seller to avoid product damage or personal injury.

NOTE: Check all parts for damage that may have occurred during transport. If such damage is found, the product must not be used. For more information, consult the seller.

#### TARGET PATIENT GROUP

People with diseases, dysfunctions, or injuries for whose treatment/rehabilitation or compensation this device is intended (see the "Intended Use" section of this manual). The product may be purchased by the user directly or on the recommendation of a physician, therapist, or other specialist. Whether purchased independently or on a clinician's recommendation, consider the available sizes/required features and variants, indications and contraindications for use, as well as the information provided by the manufacturer.

#### SAFETY

Passive lifts enable the raising of patients from a bed/wheelchair/stretchers who have significant functional limitations. Please observe the following safety guidance:

It is recommended to inspect the lift in all its parts before use.

Check the connection between the hand control and the battery.

Make sure the battery is charged.

Check that the sling shows no signs of wear or damage.

Check that the brake functions correctly.

Maximum load: 180 kg.

Caution! Do not exceed the maximum safe working load.

Caution! Always place the lift on a level surface.

Caution! Do not attempt to cross thresholds, steps, or other obstacles.

Caution! The battery may only be installed and serviced by qualified personnel.

#### ELECTROMAGNETIC INTERFERENCE

Radio waves can affect the operation of the lift. The lift may be interfered with by a mobile phone, radio, radar, etc. Please be aware of this.

#### INDICATIONS FOR USE

**PURPOSE** :The electric patient lift is intended to assist caregivers and medical personnel in safely and comfortably lifting and transferring patients with reduced mobility.

**INDICATIONS FOR USE**: It may be used:

For patients with severe mobility limitations who cannot move independently,

For individuals undergoing rehabilitation after surgery, injury, or neurological disease,

For patients with neurodegenerative disorders,

For older adults with reduced muscle strength or balance,

In the home, care facilities, hospitals, or other medical settings where safe transfers from bed to wheelchair, toilet, or other positions are required.

#### APPLICATION

The lift is used for:

Raising and lowering a patient from a bed, wheelchair, stretcher, or toilet,

Transporting a seated patient indoors,

Enabling daily care, repositioning, or hygiene procedures in an ergonomic and safe manner for both patient and caregiver.

The device is equipped with an electric actuator, safety systems (emergency stop, manual emergency lowering), a battery, and a hand control for easy operation.

#### CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product. The lift is not recommended:

For patients with a highly unstable medical condition who cannot tolerate position changes or transfers,

For individuals with severe spinal injuries requiring immobilization,

For patients who cannot be safely seated in a sling (e.g., due to open wounds, major body deformities, or significant fall risk),

For people with uncontrolled seizures during transfer,

In environments that do not meet safety requirements (slippery/uneven surfaces, insufficient maneuvering space, lack of trained staff),

In high-humidity settings—e.g., directly in the shower or bathtub.

#### COMPONENTS

Electric patient lift, user manual

A – Base

B – Support column / mast

C – Actuator

D – Base spreader lever

E – Control unit

F – Hand control (handset)

G – Lifting arm

H – Manual emergency lowering


I – Spreader bar

J – Front casters


K – Rear casters with brake

#### TECHNICAL SPECIFICATION

Dimensions: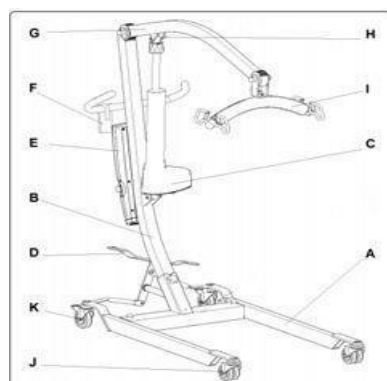

Lowest CAP point position (A): 79 cm  
 Highest CAP point position (K): 180 cm  
 Lifting range (L): 101 cm  
 Total length: 108 cm  
 Total width: 55 cm  
 Total height: 126.5 cm  
 Minimum leg spread (P): 45 cm  
 Maximum leg spread (M): 92 cm  
 Leg/frame height (E): 9.5 cm  
 Total width (closed) – external dimension (Q): 55 cm  
 Total width (open) – with wheels forward (N): 102.2 cm  
 Minimum wall clearance / CSP\* (minimum height) (C): 33.3 cm  
 Minimum wall clearance / CSP\* (maximum reach) (C1): 20.7 cm  
 Minimum wall clearance / CSP\* (maximum height) (D): 42.3 cm  
 Turning radius: 126 cm  
 Weight and load capacity:  
 Total weight: 43 kg  
 Maximum load: 180 kg  
 Maximum sling load: 150 kg  
 Reach:  
 Maximum reach at 60 cm with leg support (a1): 49.5 cm  
 Maximum reach from support point (B): 40.5 cm  
 Reach from support point at 70 cm spread (H): 51.5 cm  
 Power supply and operating conditions:  
 Supply voltage: 100–240 V  
 Operating temperature: +5°C to +40°C  
 Relative humidity: 20%–90% at 30°C, non-condensing  
 Noise level: < 50 dB(A)  
 Motor supply voltage: 24 V DC  
 Motor protection class: IP54  
 Insulation class: Class II – Type B  
 Operating functions:  
 Operating performance: approx. 40 lifts per charge  
 Duty cycle: 2 minutes continuous operation / 20 minutes pause  
 Battery capacity: Lithium battery 2.2AH  
 Emergency stop: Yes  
 Manual emergency lowering: Yes  
 Electrical emergency lowering: No  
 Material:  
 Frame: powder-coated steel  
 Wheels: plastic  
 Sling: nylon fabric

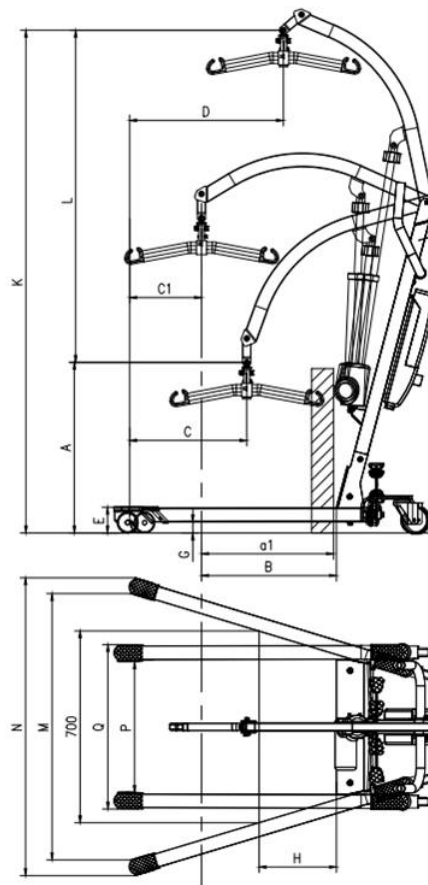This symbol indicates the maximum user weight.

Note! Actual product dimensions may vary within  $\pm 2$  cm.

#### ASSEMBLY

Assembly of the lift column

Remove the lift base (A) and lift mast (B) from the box. Align and insert the lift mast into the base.

Insert the pin and tighten the locking handle.

Note! Check that everything is securely fastened.

#### Assembly of the lifting arm

Remove the lifting arm (G) from the box, align it with the appropriate holes in the support column (B), insert it, and tighten the fixing screw using the included Allen key.


#### Assembly of the actuator

Insert the actuator (C) between the lifting arm (I) and the lift mast (B).

Then, after preparing the two latches, insert the pin of one latch into the special hole at the top of the actuator and lock it with the special latch. Repeat the same procedure for the bottom of the actuator.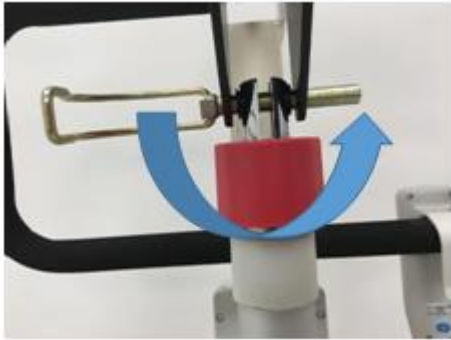

### Assembly of the handles



Remove the soft handles from the box. Then take the supplied screws and place the washers on them. Align the holes in the handles with the holes in the support column (B), insert the screws, and tighten them using the included Allen key.

Note! The handles must face inward toward the lift.

### Assembly of the base adjustment lever

Remove the base adjustment lever from the box (D). Then take the supplied screw and align the hole in the lever with the hole in the lift base (A). Insert the screw and tighten it with the included Allen key.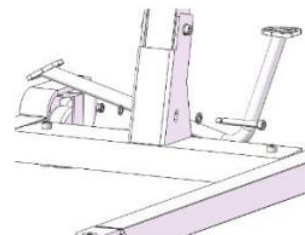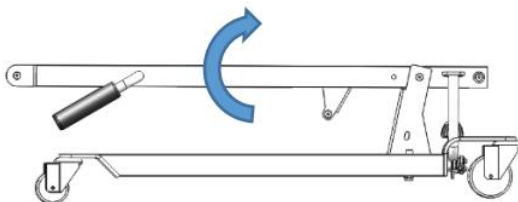

### Assembly of the control unit

Attach the control unit (E) to the support column (B) by sliding it from the top onto the metal bracket located on the column. This movement is smooth and requires no additional locks.

Note! Push it all the way down. **Note:** The outer buttons (left and right) in this model do not perform any functions.

### Inserting the battery

Insert the battery into the appropriate slot in the control unit (E).


### Connecting the actuator and hand control wires to the control unit

A) Connect the 4-pin plug to socket no. 2

B) Connect the 8-pin plug to socket no. 1

Note! Ensure that the plugs are fully inserted.

A)

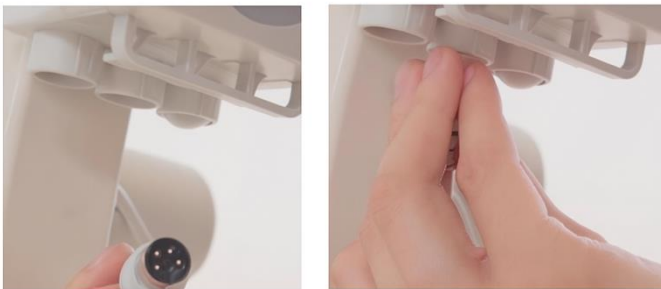B)

### Attaching the sling

Attach the head section loops to the outer hooks.


Attach the middle section loops to the inner hooks.

Attach the leg section loops to the inner hooks.

## WARNINGS AND SAFETY

### Manual emergency lowering:


When the lifting arm is raised, turn the red knob (H) clockwise to activate manual emergency lowering. To manually raise the arm, turn the red knob counterclockwise.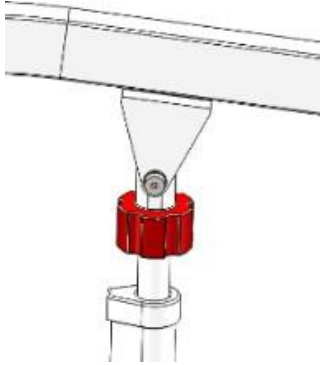

### Red emergency stop button:

Press the red emergency stop button when the lift is not in use to switch off the power supply. To restore power, turn the button clockwise.

Note! Ensure the red emergency stop button is deactivated before use.

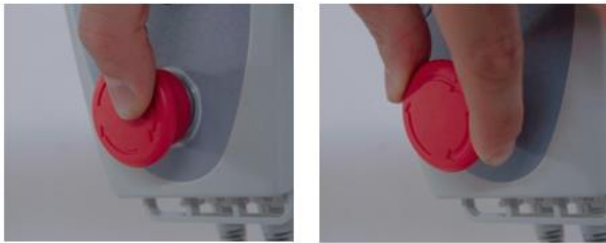### Battery charging:

Use the supplied charger and insert the plug into the appropriate socket on the side of the control unit. Then connect the black transformer to a mains outlet.

#### Battery status:

Red LED: charging in progress

Green LED: battery fully charged

Note! Always ensure the battery is charged before use. Do not charge the battery in a damp environment. Do not use the lift while charging.

### Rear wheels with locking brake:

The lift's rear wheels can be locked. Press the brake down to lock the wheels. To unlock, lift the brake back to the horizontal position.

Brakes released / Brakes locked

Note! Ensure the brake is fully engaged to effectively lock the wheels.

**Patient transport:**

When the patient is in the sling, use the remote control (F) to raise the arm:

Up arrow: raising the arm

Down arrow: lowering the arm

Note! The remote has two illuminated buttons:

1. Weight check (Red LED: limit exceeded)
2. Battery charge level (Red LED: battery nearly discharged)

**SLING INSTALLATION INSTRUCTIONS**

Hang the sling on the lift arm by attaching the upper loops to the hooks on the spreader bar.

Cross the lower sling straps under the seat, guiding the left strap to the right hook and the right strap to the left hook.

Crossing the straps creates two separate openings for the patient's legs, which stabilizes the patient's position during lifting.

Make sure the straps are not twisted and are evenly tensioned before starting the lifting process.

**Patient Transport Guidelines:**

Use the foot pedal to move the lift.

Transport the patient with the base in the open position for greater stability.

Always transport the patient facing forward.

Lock the rear wheels at every stop.

Before lifting, make sure the straps are tight and properly attached to the spreader bar.

Never leave the patient unattended in the sling.

Avoid sudden movements.

**TROUBLESHOOTING AND CORRECTIVE MEASURES**

If the device does not work, you should:

Check whether the emergency stop button has been pressed.

Check whether the hand control or actuator cables are properly connected.

Check whether the battery is discharged.

Check the battery contact panel for damage. If necessary, contact an authorized supplier.

If the device cannot be charged:

Check whether the charger is connected to the socket.

Check the battery contact panel for damage.

If the device suddenly stops during lifting:

Check whether the emergency stop button has been pressed.

Use the emergency lowering function.

If necessary, contact an authorized supplier.

**NOTE:**

In the event of a “serious incident” related to the device, which has directly or indirectly led, could have led, or may lead to any of the following events:

- a) the death of the patient, user, or another person, or
- b) a temporary or permanent deterioration in the health of the patient, user, or another person, or
- c) a serious threat to public health,

this “serious incident” must be reported to the manufacturer and the competent authority of the Member State in which the user or patient is resident.

NOTE: If pain, allergic reactions, or other concerning or unclear symptoms related to the use of the medical device occur, consult a healthcare professional.

NOTE: The product must not be used in any way other than as intended.

**MAINTENANCE AND CLEANING****MAINTENANCE**

The product requires regular maintenance. By following the guidelines in this section, you can use the product reliably for years. Some routine checks can be performed by the user, while others require assistance from the supplier. In case of any maintenance or operational issues, contact your product supplier.

Note! Like most electrical or mechanical devices, this product is sensitive to environmental conditions. Avoid damp environments. Direct contact with water or moisture may cause failure of the lift’s electrical or mechanical components. Water may corrode electronic parts, and the frame may rust!

If the product is exposed to water, perform the following steps:

Dry the product as much as possible with a cloth and place it in a dry location;

Perform all safety and functional tests before use;

If any irregularities are found, contact your product supplier.

**GENERAL GUIDELINES**

Avoid impacts or shocks to the control box, especially the hand control;

Avoid prolonged exposure to harsh conditions such as overheating, extreme cold, or humidity, and keep the control box clean;

Check all electrical connections to ensure they are secure and safe;

When all battery indicator lights are lit, this means the battery is fully charged and the controller and electrical system are functioning correctly;

If only one LED is lit, this means the battery is nearly discharged, although the electrical system is still working properly;

Check all cable connections to ensure they are secure and show no signs of corrosion;

All wheel bearings are lubricated and sealed. No additional oiling is required.

**PRODUCT INSPECTION**

Daily inspection:

Visually check that the lift is not damaged or worn.

Check that the bolts and nuts on the arm, actuator, and base are tightened properly.

Inspect the hooks and slings for damage.

Test the hand control, actuator, and width adjustment function.

Test the emergency lowering function. If in doubt, contact your supplier.

Weekly inspection:

Disconnect the hand control and actuator connectors from the control box, check for secure connections and possible corrosion. If necessary, contact your supplier.

Ensure that all bolts are secure and that no cracks or bends are present. Check that the brakes work properly.

Monthly inspection:

Check wear of the swivel castors. If necessary, contact your supplier.

Inspect the four wheels for wear and loosening – this may indicate the need for adjustment or bearing replacement. Contact your supplier for repair or replacement.

Keep the product clean – free of hair, food scraps, drinks, etc.

Temperature (some lift parts are sensitive to temperature):

Charger: At very low temperatures, the battery may freeze. The risk of freezing depends on several factors, including the type of charger, usage pattern, and battery chemistry (e.g., lead-acid or gel batteries).

High temperatures may slow down the operation of the lift. The control box protects the motor and electronic components from damage.

**CLEANING**

Never rinse the product with water or allow it to come into direct contact with water.

The product can be easily cleaned with a damp cloth.

Never use chemicals to clean the sling – this may damage and tear the material. Clean it with a damp cloth and mild soap solution, then allow it to dry completely.

**STORAGE:** The product should be stored in a dry environment, at temperatures above 0°C and relative humidity ≤60%. Do not store in extreme temperatures. For storage, remove the battery. Failure to meet these conditions may cause rusting of the lift and damage to the electrical system.

**DISPOSAL:** The product contains electronic components and must not be disposed of with household waste after the end of its service life. For proper disposal, it should be taken to an appropriate collection point for electrical and electronic waste, operated by local authorities or vendors offering this service. Proper disposal prevents potential harm to the environment and human health and enables material recovery, contributing to energy and resource savings in line with applicable legal regulations on recycling of electrical and electronic equipment.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. ANTAR MEDICAL Sp. z o.o., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR MEDICAL Sp. z o.o.  
03-068 Warsaw, ul. Zawiślańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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